Manufacturer	Výrobce	Tootja
Fabricante	Κατασκευαστής	Ražotājs
Hersteller	Gyártó	Gamintojas
Fabricant	Prodútor	Producēt
Fabbricante	Výrobca	Изоготовитель
Fabrikant	Üretici	Proizvodač
Valmistaja	Fabricante	Izdeļotālec
製造商	Tillverkare	יצרן
製造元	Fabrikant	제조업체
Производитель	الجهة المصنعة	Nhà sản xuất
Proizvodač	Producēt	

Date of Manufacture	Datum výroby	Tootmiskuupäev
Fecha de fabricación	Ημερομηνία κατασκευής	Izgatavošanas datums
Herstellungsdatum	A gyártás időpontja	Pagaminimo data
Date de fabrication	Data producerii	Data produkcji
Data di produzione	Dátum výroby	Дата изготовления
Vervaardigingsdatum	Üretim Tarihi	Datum proizvodnje
Valmistuspäivämäärä	Data de fabrico	Datum izdelave
製造日期	Tillverkningsdatum	רציי יראת
製造年月日	Fabrikasjonsdato	제조일자
Дата на производство	تاريخ التصنيع	Ngày sản xuất
Datum proizvodnje	Fremstillingsdato	

Medical Device	Orvostechnikai eszköz	Медицинское устройство
Producto sanitario	Dispozitiv medical	Medicinski uređaj
Medizinisches Gerät	Zdravotnicka pomůcka	Medicinski pripomoček
Dispositif médical	Medikal Cihaz	יאופר רשכח
Dispositivo médico	Dispositivo médico	의료기기
Medisch hulpmiddel	Medicinsk utrustning	Thiét bǐ y tié
Lääkinnällinen laite	Medisinsk enhet	
医療器械	جهاز طبي	
Медицинско изделие	Medicinsk enhed	
Medicinski uređaj	Medicinsisade	
Zdravotnický prostředek	Medicinska ierice	
Ιατρική Συσκευή	Medicinos prietais	
	Urządzenie medyczne	

Use by Date	Χρησιμοποιήστε μέχρι την	“Izlietot līdz” datums
Fecha de caducidad	Ημερομηνία	Galiojimo data
Verfallsdatum	Felhasználhatósági idő	Data przydatności do użycia
Date limite d'utilisation	A se folosi până la data	Использовать до указанной даты
Da utilizzare entro la data indicata	Použite do dátumu	Datum upotrebe
Uiterste houdbaarheidsdatum	Son Kulanma Tarihi	Datum uporabnosti
Viimeinen käyttöpäivä	Prazo de validade	שומישל וורא יראת
此日期前使用	Bör användas före	사용 기한
使用期限	Brukes innen	Hạn sử dụng
Срок на годност	سخدم قبل التاريخ	
Upotrijebiti do naznačenog datuma	Holdbarhedsdato	
Datum pouzitelnosti	Tarvitada enne kõlblikkustähtpäeva	

Importer	Εισαγωγέας	Importuotojas
Importador	Importör	Importer
Importeur	Importator	Импортер
Importateur	Dovozca	Uvoznik
Importatore	İthalatçı	Uvoznik
Importeur	Importador	יבואן
Maahantuoja	Importör	수입 업체
进口商	Importør	Nhà nhập khẩu
輸入業者	المستورد	
Вноситель	Importer	
Uvoznik	Maaletooja	
Importér	Importētājs	

Medical Product Services GmbH
Borngasse 20, 35619 Braunfels Germany
info@mps-gmbh.eu

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90119 Version 2 2026-02









MiniMe® 2 Nasal Vented Mask

These instructions contain the following languages:

English, Spanish, German, French, Italian, Dutch, Finnish, Chinese, Japanese, Bulgarian, Croatian, Czech, Greek, Hungarian, Romanian, Slovak, Turkish, Portuguese, Swedish, Norwegian, Arabic, Danish, Estonian, Latvian, Lithuanian, Polish, Russian, Serbian, Slovenian, Hebrew, Korean and Vietnamese

Sleepnet Corporation
5 Merrill Industrial Drive
Hampton, NH 03842 USA
Tel: (603) 758-6600
Fax: (603) 758-6699
Toll Free: 1-800-742-3646 (USA)
www.sleepnetmasks.com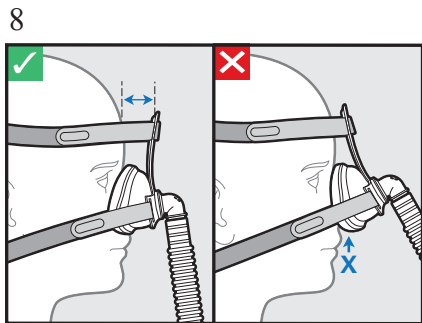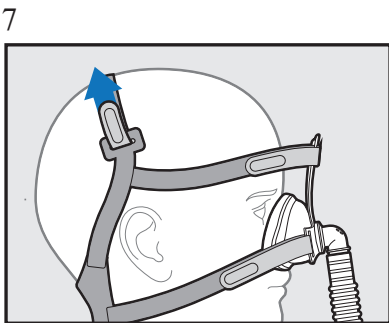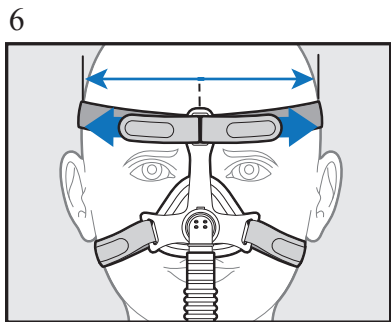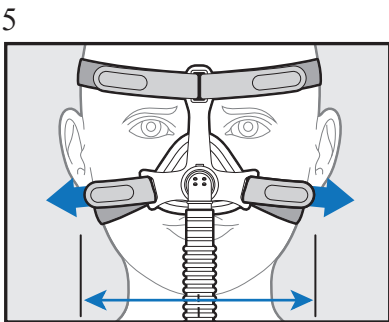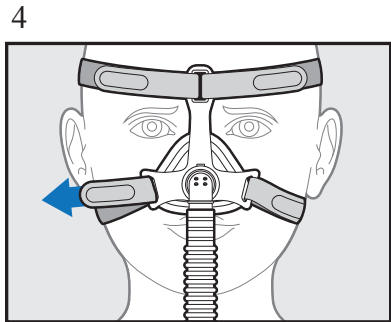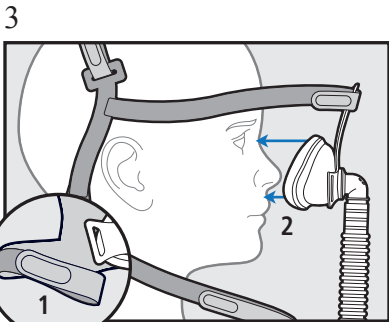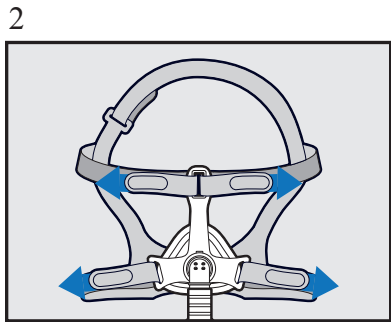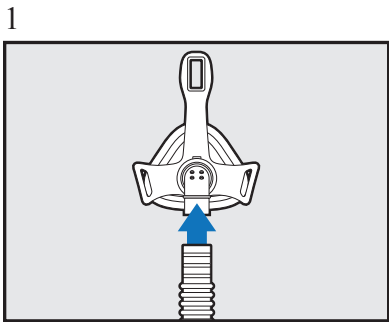

Explanation of Symbols Which Appear on the Device Labeling
Explicación de símbolos que aparecen en la etiqueta del dispositivo
Erklärung der Symbole auf dem Etikett dieses Gerätes
Explication des symboles qui apparaissent sur l'étiquette de l'appareil
Spiegazione dei simboli che figurano nelle etichette del dispositivo
Toelichting bij symbolen die op de etiketten van het apparaat staan
Laitteen etiketissä olevien symbolien selitykset
装置标签上的符号说明
装置のラベルにある記号の意味
Обяснение на символите върху етикетите на устройството
Objasnjenje simbola koji se javljaju na etiketi uređaja
Vysvetlení symbolů uvedených na štítku přístroje
Επεξήγηση των συμβόλων που εμφανίζονται στην επισήμανση της συσκευής
Az eszköz címkézésén szereplő szimbólumok magyarázata
Explicatia Simbolurilor Care Apar pe Eticheta Dispozitivului
Vysvetlivky k symbolom, ktoré sa objavujú na výrobných štítkoch zariadení

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MiniMe® 2 Nasal Vented Mask

CONTENTS: MiniMe® 2 Nasal Vented Mask, tubing, and headgear
THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX
Rx ONLY MADE IN U.S.A.

The MiniMe 2 Pediatric Nasal Vented Mask is intended to provide a patient interface for application of positive pressure therapy. The mask is to be used as an accessory to CPAP or Bi-level positive pressure systems. It is intended for patients >2 years old and <12 years old. It is intended for single patient, multi-use in the home, hospital, or institutional environment.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the mask to temperatures above 122°F (50°C).

WARNING: Do not use the Mask with baseline pressure less than 4 cm H₂O or greater than 20 cm H₂O.

CAUTION: The mask is designed to provide a stable, effective air seal with minimum headgear tension. Excessive headgear tension may lead to facial deformities with long-term use. Regular clinical review is recommended.

CAUTION: If skin irritation or any other adverse reaction develops, discontinue use and contact your health care professional.

CAUTION: This mask may be cleaned and used repeatedly on the same person. Do not sterilize.

WARNING: This mask is not suitable for providing life support ventilation.

WARNING: Do not block or try to seal the vent holes.

WARNING: At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask is intended for use on children >2 years of age and <12 years of age. The mask is designed to seal over the child's nose. Ensure that the nasal mask cushion does not block the nostrils at any time during fitting or use.

WARNING: Children should be attended at all times during therapy.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Consult manufacturer for proper O₂ enrichment tubing. At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.

WARNING: Using a mask may cause tooth, gum, or jaw soreness or aggravate an existing dental condition. If symptoms occur, consult your physician or dentist.

WARNING: Significant unintentional leak may occur without properly fitting the mask. Follow the USER INSTRUCTIONS FOR MINIME 2 NASAL MASK provided.

WARNING: If the mask is prescribed for children with serious congenital conditions or cognitive impairments then the positive pressure device must have the appropriate alarms. The device, mask, and alarms must be tested before starting therapy.

GENERAL INFORMATION

The mask has a non-transferable three month limited warranty for manufacturer's defects from the date of purchase by the initial consumer. If a mask used under normal conditions should fail, Sleepnet will replace the mask or its components. For additional information about the Sleepnet Warranty, please visit www.sleepnetmasks.com. Per EU MDR 2017/745, please report serious incidents to manufacturer and EU Member State competent authority.

NOTE: The mask has a service life of 6 months.

NOTE: Please dispose of the mask and accessories in accordance with local and regional guidelines.

NOTE: THIS PRODUCT IS NOT MANUFACTURED WITH NATURAL RUBBER LATEX.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Hand wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing child's face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.90 cmH₂O

Resistance to air flow at 100 L/min: 3.60 cmH₂O

Dead space (approx): **XS size-** 20 ml **S size-** 25 ml **M size-** 37 ml **L size-** 40 ml

A-weighted Sound Pressure Level at 1 m distance: 29.89 dBA; A-weighted Sound Power Level: 32.93 dBA

Operating Temperature: Do not expose the mask to temperatures above 122°F (50°C)

USER INSTRUCTIONS FOR THE MINIME® 2 NASAL MASK

1. Attach the tubing assembly to the MiniMe 2 Mask (Figure 1).
2. Lay headgear flat with light gray side facing up with the head strap at the top as shown in Figure 2. Place mask gel side down in the center of the headgear and thread all 4 straps through the slots as shown (Figure 2).
3. To place mask on child, detach one or both lower straps from the mask. Slip the headgear over the head. Seat the bottom cushion of the mask between the upper lip and nose, and tip the mask up to contact the nose (Figure 3).

NOTE: The Touchless Spacebar™ should not contact the forehead.

4. Find the detached hook and loop fastener strap and reattach (Figure 4).
5. Ensure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head.
6. Adjust the bottom headgear straps by using the hook and loop fastener tabs (Figure 5).
7. Adjust the upper headgear straps by using the hook and loop fastener tabs: detach the hook and loop fastener tabs then pull straps forward to center the mask over face. Ensure headgear is centered by aligning the two straps. Then pull straps back, adjust strap tension, and affix tabs (Figure 6).
8. Adjust the head strap by using the hook and loop fastener tab (Figure 7).
9. If mask is uncomfortable repeat steps 6-9.

NOTE: Do NOT overtighten the headgear.

NOTE: The Touchless Spacebar™ should NOT touch the forehead. It should be parallel to the forehead (Figure 8).

NOTE: For optimal seal and comfort, mask should be snug, but not tight. Adjust the tension by detaching the straps, tightening or loosening the straps, and reattaching the hook and loop fastener tabs.

10. Connect the mask tubing assembly to the positive airway pressure device tubing and turn the device on to the prescribed pressure (Figure 9).
11. Gently press the edges of the mask to change mask shape, create a comfortable fit, and reduce leaks.

NOTE: The MiniMe 2 Nasal Mask contains Custom Fit Technology™. This soft mask shell can be shaped to follow the contours of a child's face. The Memory Wire™ embedded into the shell locks the customized fit in place.

12. Turn head from side to side while lying on the bed and identify areas of leak that may prevent sleep.
13. If leaks are noticed at the bridge of the nose, slightly tighten the top straps as described in in step 7. The Touchless Spacebar™ should not contact the forehead.
14. If leaks are noticed at the upper lip or below the nose, slightly tighten the bottom straps.

MiniMe® 2 is a trademark of Sleepnet Corporation.

Visit our website at www.sleepnetmasks.com.

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